This document is to inform the cleaning methods of Serres canisters and the angle connectors of Serres canisters.

CLEANING OF SERRES CANISTERS

- Serres Canisters are made of polycarbonate
- as cleaning method we recommend
 - wiping with a towel or
 - washing in a washing machine in + 95°C
- Serres canisters can be autoclaved in + 121°C

NOTE!

- before autoclaving the canisters should be rinsed out of the disinfection detergent
- when following the instruction canisters can stand approx. 100 autoclaving

CLEANING OF ANGLE CONNECTOR OF SERRES CANISTER

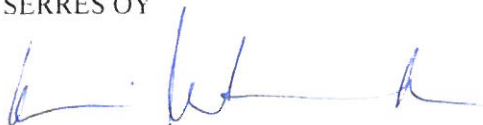
- angle connector of the Serres canister is made of TPE
- as cleaning method we recommend
 - cleaning with a towel or
 - washing in a washing machine in + 95°C
- the angle connector of the Serres canister can be autoclaved in + 121°C

NOTE!

- angle connector should be disconnected from the canister when washing and autoclaving
- washing and autoclaving will shorten the time of usability, products should be changed to new ones if they don't pass the vacuum test

Kauhajoki 25.6.2013

SERRES OY

A handwritten signature in blue ink, appearing to read "Virpi Metsäranta".

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